


Instructions For Use:

After setting up as per Assembly Instructions

Step 1 Load Cow
Walk Cow into Wrangler
Attach Rump Chain
Close Head bail
Attach front Belly Straps (as required)



Step 2 Lower Calving Attachment
Swivel Calving Attachment down
Slot  Pipe Inserts into Side Bar Ring
and secure with 4mm R Clip.
Repeat on other side.



Step 3 Pull Calf
Rope attaches to pin on back leg winch,
round back leg bar then onto calf
Tension using back winch (remove handle from holder)
and pressure on rope.
N.B. Be careful not to use too much tension on cow.



Attachment has two height/angle settings.
Use top Side Bar ring for level pull
Use bottom Side Bar ring for downwards pull.