

Instructions For Use:


After setting up as per Assembly Instructions


Step 1 Load Cow

Walk Cow into Wrangler
Attach Rump Chain
Close Head bail
Attach front Belly Straps (as required)



Step 2 Lower Calving Attachment

Swivel Calving Attachment down
Slot  *Pipe Inserts* into *Side Bar* Ring
and secure with *4mm R Clip*.
Repeat on other side.

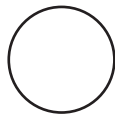

Step 3 Pull Calf

Rope attaches to pin on back leg winch,
round back leg bar then onto calf
Tension using back winch (remove handle from holder)
and pressure on rope.
N.B. Be careful not to use too much tension on cow.

Attachment has two height/angle settings.
Use top *Side Bar* ring for level pull
Use bottom *Side Bar* ring for downwards pull.